SuperBar™

SPB-S
SPB-HD
SPB-AK
(DC202 - RevA)

healthcraftproducts.com

⚠️ WARNING FOR BED SAFETY

⚠️ WARNING

ENTRAPMENT HAZARD


There is a risk of entrapment associated with all bedside mobility aids. Entrapment occurs when an individual becomes wedged between a bed rail and another object (example: mattress). Entrapment can result in serious injury and death. Read and understand the enclosed Entrapment Prevention Guidelines document. See the additional Entrapment Prevention Guidelines:

- www.fda.gov/bedsafety
- <https://www.cpsc.gov/safety-education/safety-guides/furniture-furnishings-and-decorations/adult-bed-rails>


PRIOR TO INSTALLATION, review all of the product specifications and instructions to ensure that it is safe to use with your bed and mattress. **Stop using immediately if product or bed shifts from its original position, if product is damaged, broken or missing parts.**

PRIOR TO EVERY USE, ensure the pole is installed correctly in one of the specified locations:

(a) No gap between the pole and the side of the mattress. Ensure a minimum distance 4.75"/121mm between horizontal bar and mattress height.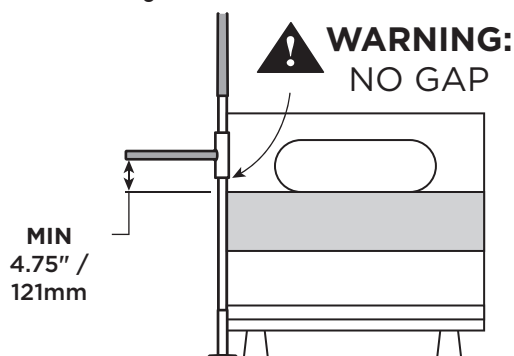

(b) A gap of at least 12.5"/320mm between the pole and the side of the mattress. Ensure a minimum distance 4.75"/121mm between horizontal bar and mattress height.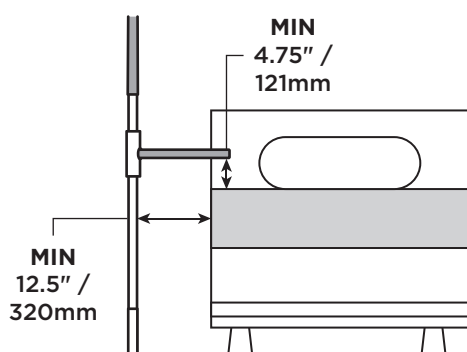

⚠️ WARNING


ENTRAPMENT HAZARD

NEVER use product without properly securing beside the bed. Incorrect installation can allow product to move away from mattress, bed frame and/or head or foot boards, which can lead to entrapment and death.

⚠️ WARNING

ENTRAPMENT HAZARD

(a) There is an increased risk of entrapment when using a safety product beside a bed with a soft mattress. Test mattress by applying 12lbs (5.4kg) of weight on the edge of the mattress. If the mattress can compress by 2" (51cm), do not use this product.

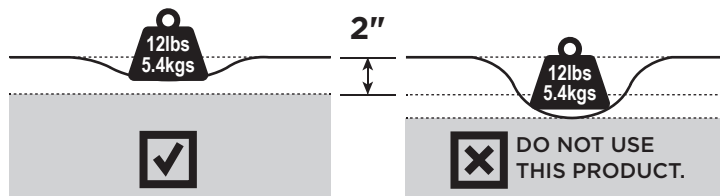(b) If product is used in a care facility, follow the installations policies of the institutions. If this product does not comply with those policies, do not install this product.

(c) When changing bed linens, always confirm that product is installed correctly as per installation instructions. Re-position to eliminate a gap between product and side of mattress.

For questions about product safety or installation, or in case of a serious event with this product, contact a the HealthCraft at 1-888-619-9992 immediately. In addition to contacting the manufacturer directly, consumers should report problems to the CPSC at this website: <https://www.saferproducts.gov/> or call 1-800-638-2772, or to the FDA at 1-800-332-1088.

⚠️ WARNING

SUFFOCATION, STRANGULATION, ENTRAPMENT HAZARD

If product is installed incorrectly or moves from its initial position gaps can occur which can entrap and cause fatality. People with Alzheimer's disease, dementia or other neurological conditions, or those who are sedated, confused, or frail, are at increased risk of entrapment, suffocation and strangulation.

- **NEVER** use unless product is tight against mattress, without gaps, and at least 12-1/2 in. from headboard and footboard.
- **NEVER** use with children.
- **NEVER** use on toddler, bunk, water, electrical or inflatable beds, or on beds with mattress toppers or soft compressible pads.

This product is under the jurisdiction of the FDA; to report problems or injuries, call 1-800-332-1088.

⚠️ WARNING

ENTRAPMENT, STRANGULATION, SUFFOCATION AND FALL HAZARDS

Gaps in and around this product can entrap and kill. People with Alzheimer's disease or dementia, or those who are sedated, confused, or frail, and are at increased risk of entrapment and strangulation. People attempting to climb over this product are at increased risk of injury or death from falls. Always make sure this product is properly secured beside the bed. If product can move away from bed or mattress, it can lead to entrapment and death.

English

OVERVIEW

Thank you for choosing the SuperPole System from HealthCraft. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your SuperPole System is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your SuperPole System installed by a qualified contractor.

The SuperPole System is intended for moderate vertical load bearing to provide sitting and standing support for people with reduced mobility. The floor to ceiling range and maximum user weight dependent on model (see Page 16 for details and product / accessory compatibility chart). Product is not intended to support full body weight. The SuperPole System is not to be used in any other way than described above.

The SuperPole is a high quality floor to ceiling safety pole. The SuperPole must be installed between a structural floor and ceiling by turning a jackscrew at the bottom of the pole. The pressure created by the jackscrew is strong enough for the SuperPole to support the user, yet it does not require drilling holes into the floor or ceiling. This is ideal for relocating or removing the pole. NOTE: The top plate has two holes that may be used to fix the plate directly to the ceiling joist if necessary. Do not install the pole where it is directly exposed to water. See Page 10 for details.

The SuperBar is a high quality pivoting and locking horizontal bar that connects to the SuperPole (floor to ceiling safety pole). The horizontal bar can be lifted, pivoted and lowered to the next locking position around the SuperPole. In any of the 8 lowered position (every 45 degrees) the SuperBar can be used to support the user. It is important to set proper height and orientation of the SuperBar to optimize the user's range of motion.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, rusting due to water exposure, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice. Manufacturer: HealthCraft Products Inc. 2790 Fenton Road, Ottawa, Canada, K1T 3T7. © 2024 HealthCraft Products Inc.

DELIVERY CONTENTS - SUPERPOLE

FIGURE A.

- | | |
|--------------------------------------|------------------------|
| 1. Pole | 8. Top Plate Assembly |
| 2. Pole Grip, 20" length | 9. Base Plate Assembly |
| 3. Cover Sleeve (with warning label) | 10. Nylon Washer |
| 4. Threaded Collar | 11. Cotter Pin |
| 5. Jam Nut | 12. Clevis Pin |
| 6. Extension Screw | 13. C Clip |
| 7. Plastic Washer | |

DELIVERY CONTENTS - SUPERBAR

FIGURE A.

- | | |
|--------------------------------------|--------------------------------|
| 14. T-Bar Assembly | 18. #8-32 Flat Head Screw (x2) |
| 15. Spring Ring (with warning label) | 19. Pivot Bushing (x2) |
| 16. Castellated Collar | 20. 3/32" Hex Key |
| 17. 5/16-24 Set Screw (x3) | 21. 5/32" Hex Key |

TECHNICAL DATA

See Figure B.

INSTALLATION WARNINGS

FIGURE C.

1. Use caution while maneuvering pole during installation.
2. Standard pole can be installed directly under 1 ceiling joist.
3. Standard pole can be installed directly under 2 ceiling joists.
4. Heavy Duty (HD) pole must be secured to ceiling joist (s).
5. If subject to heavy usage, pole must be secured to ceiling joist (s).
6. CAUTION - Damage may occur if extension screw is overtightened.
7. CAUTION - Do not install if the structure above ceiling is unknown.
8. CAUTION - Do not install on loose floor coverings.
9. CAUTION - Do not install in rooms with radiant ceiling heat.
10. CAUTION - Do not install on angled ceilings.
11. CAUTION - Do not install the pole where it is directly exposed to water.

SUPERPOLE PLACEMENT

FIGURE D.

1. BEDROOM - Locate pole at least 12.5"/320mm from headboard or footboard, adjacent to waistline and either (a) against the mattress - no gaps, or (b) 12.5"/320mm away from mattress.
2. BATHTUB - Locate pole base against tub, approximately half way along its length.
3. SEATING (TOILET or CHAIR) - To provide clearance when standing, locate pole 3-10" / 76mm-254mm forward of the knees, and 2-6" / 51mm-154mm to the side of knees when sitting.

SUGGESTED SUPERBAR POSITIONS

FIGURE D.

- | | |
|---|--|
| 4. BEDROOM | 5. BATHROOM or 6. SITTING |
| A = side support / out of the way | A = side support / out of the way |
| B = in-bed support | B = standing or transfer support |
| C = sitting support | C = standing support |
| D = standing support | D = standing or transfer support |
| E = transfer support to walker / wheelchair | E = standing or transfer support |
| | F = transfer support in/out of bathtub |

SUPERPOLE+SUPERBAR INSTALLATION INSTRUCTIONS

FIGURE E.

1. Slide spring ring upwards on castellated collar to expose 3 set screws.
2. Back off 3 cone point set screws with hex key so castellated collar can slide onto pole without set screws contacting pole.
3. Install pivot bushing into T-bar assembly.
4. Align hole and install screw with hex key.
5. Remove cover sleeve from bottom of pole.
6. Slide items onto bottom of pole (note order and orientation): T-Bar assembly with installed pivot bushing top/first, spring ring with castellated collar, pivot bushing and cover sleeve.
7. Determine the height of the T-bar on the pole. When installing beside a bed, measure the height of the bed (floor to top of mattress) and add MIN 4.75"/121mm. This is the installation height of the castellated collar.
8. Install castellated collar to pole by tightening 3 set screws at this height with hex key. The set screws MUST be tightened until the back of the screw is flush with the collar or injury may result.
9. Slide spring ring downwards on the castellated collar to retain set screws.
10. Lower T-bar assembly onto castellated collar.
11. Install bottom pivot bushing into T-bar assembly.
12. Align hole and install screw with hex key.
13. On a flat horizontal surface, place top plate onto pole.
14. Align holes and install clevis pin with washer on opposite side.
15. Install cotter pin into clevis pin.
16. Place base plate on floor at intended location, and tilt pole.
17. When upright, install the pole into the base plate hole.
18. Insert shaft of screwdriver thru drive hole of extension screw.
19. Hold pole and turn screwdriver clockwise to raise pole.
20. Top plate must contact ceiling squarely.
21. Continue turning extension screw approx 2-5 turns until pole is secure.
22. Turn jam nut until it is tight against bottom of pole.
23. Turn both jam nut and extension screw together to jam tight.
24. Remove screwdriver from extension screw.
25. Cover exposed hole on the extension screw with C clip.
26. Lower cover sleeve to conceal screw.

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