


SIZING GUIDE

		MFR #	SIZE	CALF CIRCUMFERENCE	LENGTH	TOP COLOR	TOE PORT COLOR
KNEE	REGULAR	84-01M	Small	Up to 12 in	Up to 16 in	White	Yellow
		84-02M	Medium	12 in to 15 in	16 in to 17 in	White	White
		84-03M	Large	15 in to 17 in	17 in to 18 in	White	Blue
		84-04M	X-Large	17 in to 19 in	18 in to 19 in	White	Green
	LONG	84-11M	Small	Up to 12 in	Up to 18 in	Blue	Yellow
		84-12M	Medium	12 in to 15 in	18 in to 19 in	Blue	White
		84-13M	Large	15 in to 17 in	19 in to 20 in	Blue	Blue
		84-14M	X-Large	17 in to 19 in	20 in to 21 in	Blue	Green
THIGH	SHORT	84-21M	Small	Up to 12 in	Up to 29 in	Yellow	Yellow
		84-31M	Medium	12 in to 15 in	Up to 29 in	Yellow	White
		84-41M	Large	15 in to 17 in	Up to 29 in	Yellow	Blue
	REGULAR	84-22M	Small	Up to 12 in	29 in to 33 in	White	Yellow
		84-32M	Medium	12 in to 15 in	29 in to 33 in	White	White
		84-42M	Large	15 in to 17 in	29 in to 33 in	White	Blue
		84-44M	X-Large	17 in to 19 in	29 in to 33 in	White	Green
	LONG	84-23M	Small	Up to 12 in	33 in to 36 in	Blue	Yellow
		84-33M	Medium	12 in to 15 in	33 in to 36 in	Blue	White
		84-43M	Large	15 in to 17 in	33 in to 36 in	Blue	Blue
		84-45M	X-Large	17 in to 19 in	33 in to 36 in	Blue	Green

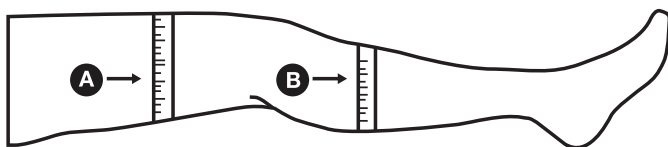

FOR PROPER FIT – KNEE LENGTH

Size based on: Circumference of “B”
Length based on: Distance from bend of knee to bottom of heel

FOR PROPER FIT – THIGH LENGTH

Size based on: Circumference of “A” and “B”
Length based on: Distance from “A” to bottom of heel

LAUNDERING GUIDELINES

Machine wash in lukewarm water. Air dry/tumble dry on low setting. Do not iron.