

HOW TO INSTALL YOUR



Tub Grab Bar

Item # 8220/8220-R


Capacity not to exceed 250 lb



No tools required
Quick Assembly

Easy to follow
step-by-step
instructions

- 1** Loosen the clamp by turning the three-pronged knob counterclockwise and remove the styrofoam block from between the clamp.
- 2** Adjust clamp opening to fit over the side of the bathtub (**Figure A**). Set the Tub Grab Bar to the location that most conveniently assists the user to enter and exit the bathtub. Tighten knob clockwise until the Tub Grab Bar is securely clamped onto the side of the bathtub (**Figure B**).

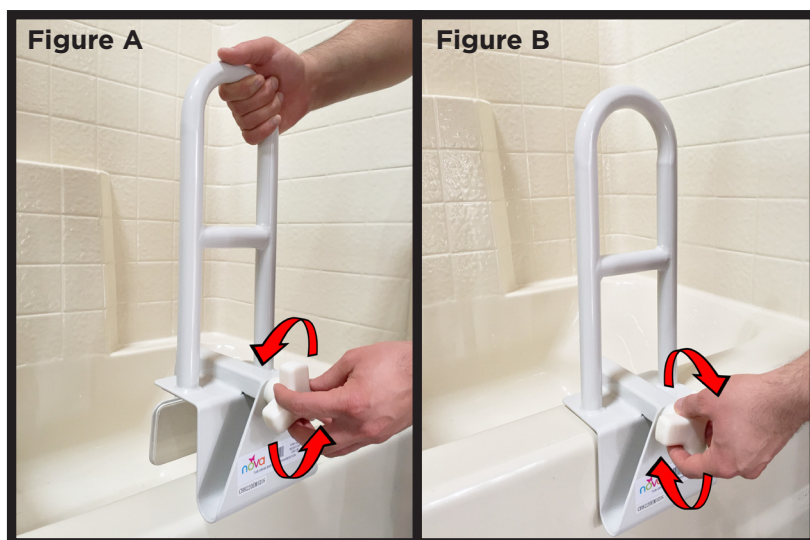

- 3** Pull on the Tub Grab Bar to make sure it is secure on the bathtub.

HOW TO USE

Hold on to the Tub Grab Bar when entering and exiting the bathtub. Use the Tub Grab Bar for balance when needed.

CARE AND MAINTENANCE

Check regularly to make sure the Tub Grab Bar is securely clamped to the bathtub. Use a mild detergent and warm water to clean.



CAUTION

Do NOT overtighten on fiberglass bathtubs. Doing so may cause bathtub to crack or break, resulting in injury or fall.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame - Limited Lifetime Warranty

Non-Metal/Metal parts - One Year Limited Warranty
(rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and its parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

- Use **EXTREME** caution when using product with wet tub or floor
- Users with limited physical capabilities should be supervised or assisted when using the Tub Grab Bar
- The Tub Grab Bar provides support for an individual up to **250 lb**. This product is not intended to support the full weight of an individual.

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE

Connect with us!

novamedicalproducts.com
facebook: [novamedicalprod](#)
twitter: [novamedicalprod](#)
twitter: [novawalkerstalk](#)
YouTube: [NOVA Medical Products](#)