

HOW TO ATTACH YOUR



SUCTION CUP ASSIST BAR 12"
Item # 8212-R



1

Select a smooth, flat, non-porous mounting surface with a completely smooth 3.5" diameter area for both suction sections. Clean the surface with a damp cloth and dry thoroughly (see figure 1).

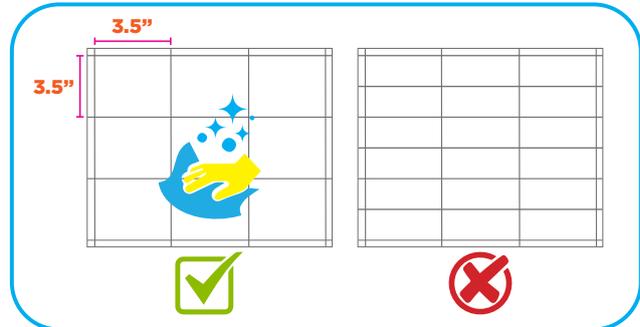


Figure 1

2

Release suction levers by pulling them up and away from the handle (see figure 2)



Figure 2

3

Firmly press the suction cups onto the mounting area and push the levers down. Be sure the safety indicator fully shows green (see figure 3).

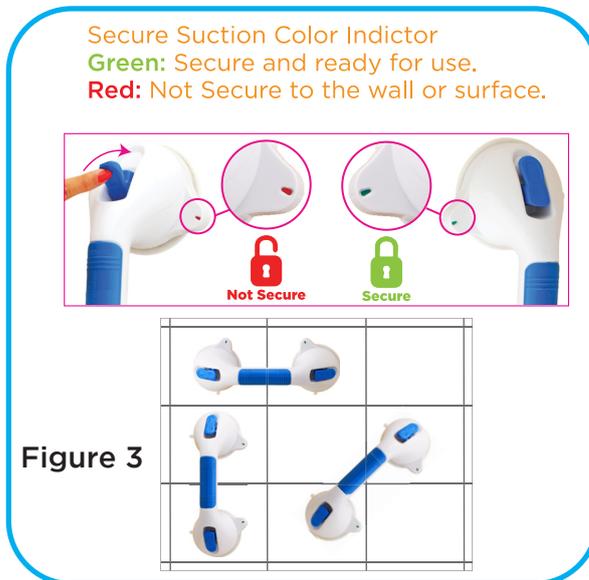


Figure 3

NOTE: Test suction strength by pulling on the handle. If loose, repeat steps 1-3 (see figure 4).



Figure 4

4

To Remove:
Flip both levers up and lift away from the wall (see figure 2). Use suction cups tabs to help release from mounting surface (see figure 5).



Figure 5

WARNING

Read all instructions carefully and consult with a qualified professional before installing to avoid possible injury. **NOVA Medical Products shall have no liability for any damage or injury resulting from improper assembly or misuse of this product.**

- Balance assist only
- Always make sure the grab bar is securely attached before each use and always reattach if loose
- Only mount to smooth, flat, non-porous surface
- Clean and dry both suction cups and mounting surface before first use.
- Re-attach the grab bar and check the safety indicators before each use
- DO NOT use for leverage or to support body weight

CARE AND MAINTENANCE

Clean the surface with a damp cloth and dry thoroughly

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame

Non-Metal/Metal parts

Limited Lifetime Warranty

One Year Limited Warranty

(rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

novajoy.com

Connect with us!

novajoy.com

 [novamedicalprod](https://www.facebook.com/novamedicalprod)

 [novamedicalprod](https://twitter.com/novamedicalprod)

 [novawalkertalk](https://twitter.com/novawalkertalk)

 [NOVA Medical Products](https://www.youtube.com/channel/UCvXp1t1t1t1t1t1t1t1t1t1)

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE