

HANDHELD SHOWER HEAD

QUICK START GUIDE

WHAT'S INCLUDED

- 1x Mounting Bracket
- 1x Hose
- 1x Shower Head
- Thread Tape Roll

OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Handheld Shower Head as easy as possible. Check out the included link and QR code to help you through the process.




To see all FAQs
in one place,
visit vhealth.link/8beba


INSTALLATION INSTRUCTIONS

1. Remove the old shower head and clean the existing threads off the water pipe. Then apply a new layer of thread, 3-4 times around the pipe is enough.
2. Before installing the mounting bracket to the existing water pipe, make sure that the two parts of the mounting bracket are connected tightly.


Make sure this connection is tight
before installing

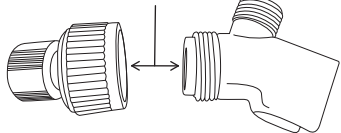


3. Install the mounting bracket to the existing water pipe.

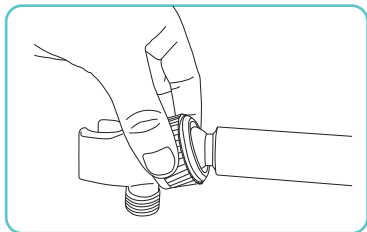

4. Connect the end of the shower hose to the holder bracket. For a tighter seal, it's recommended to add 2-3 layers of thread tape to the bottom end of the mounting bracket.

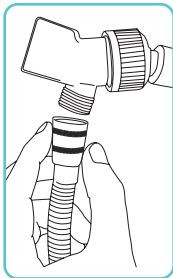

5. Attach shower head to the hose.

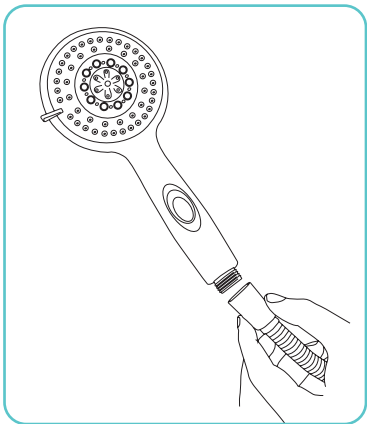

6. Make sure **all** connections are tightened down and test installation with **all** spray settings before use.

vive®

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com
Made in China