



VIVE ICE THERAPY MACHINE

RHB1049GRY
HCPCS: E0218

Designed to provide immediate relief from soreness, pain, and discomfort, the ice therapy machine includes a universal therapy pad to effectively target the ankle, knee, hip, shoulder, and arm. Alleviates swelling, pain, and inflammation following surgery or injury, the quiet ice therapy system is also useful for relieving joint pain due to arthritis or overuse.

SPECIFICATIONS

What's Included:

- Cold therapy unit
- Universal therapy pad
- Insulated connector hose
- Power cord
- Adjustable strap- small
- Adjustable strap- medium
- Adjustable strap- large
- One-year guarantee

Uses:

- Relieves post-surgery swelling and inflammation.
- Reduces post-workout stiffness and pain.

Materials:

- Durable composite, Insulated foam tubing, PVC

Power:

- Input: 110-240V AC
- Output: 12.0V-1.0A (1000mA)
- 50/60 Hz

Call now!

or send us an e-mail
now to find out more!



portal.vivehealth.com



(239) 220-5772



sales@vivehealth.com



TARGETED COLD THERAPY

Soothing cold therapy reduces pain, swelling, stiffness, and inflammation due to surgery, injury, post-workouts, arthritis, and more.

VERSATILE THERAPY PAD

Uniquely designed therapy pad contours to the body for targeted relief with three sizes of non-slip straps for optimal positioning.

THREE ADJUSTABLE STRAPS

Secure the therapy pad on the shoulder, arm, back, hip, knee, calf, or ankle, with one of the three adjustable straps.

QUIET AND COMPACT

Exceptionally quiet, the compact cooling unit holds 6L and includes a carry handle for easy portability.

INSULATED CONNECTION TUBE

Extra-long 5-foot tubing is insulated to maintain the water temperature and secures the unit with leak-proof snap clips.



PRODUCT SPECIFICATIONS

► How to Use:

- Add ice to the designated ice fill line on the base, then add cold water to the designated water fill line
- Secure the therapy machine lid on the base
- Insert the insulated hose connection in the unit until there is an audible click, then attach the therapy pad to the insulated hose, ensuring there is an audible click
- Connect the power cord to the unit and a grounded outlet
- Press the power switch located on the back of the unit lid
- Press and hold the 'M' button, then use the up and down arrows to select the rate of flow and duration of the treatment session. Press 'M' to confirm the selection
- Secure the universal therapy pad with the adjustable bands
- Press the start button



(239) 220-5772



sales@vivehealth.com

vive
health

SR23- SR1