

STAND ALONE TOILET RAIL

LVA1023

vive
vivehealth.com




To see all FAQs in
one place,
visit vhealth.link/24771

OWNER'S MANUAL

TOILET SAFETY RAIL

WHAT'S INCLUDED

- 2x Frames with Armrests (A1-A2)
- 2x U-Shape Support Bars (B1-B2)
- 1x Magazine Basket (C)
- 8x Hand Knob Screws (D1-D8)

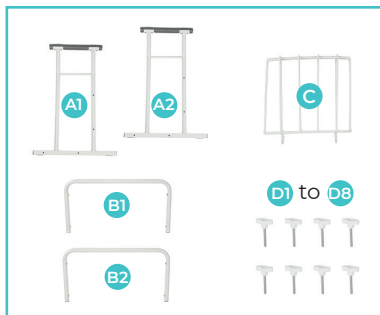


ASSEMBLY INSTRUCTIONS No Tools Required!



For an easy instructional video
Check out vhealth.link/d319f

1. Attach (B1) U-Shaped Support Bar to the base of frame with (A1) Armrest using 2 (D) Hand Knob Screws. Ensure the exposed screwholes on the vertical section of (A1) are closest to you, and the curves of (B1) are closer to the center of (A1) 's base.

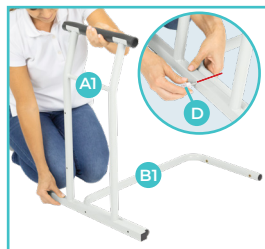

2. Attach the other frame with (A2) Armrest to the other side of the (B1) U-Shape Support Bar. Fasten with 2 (D) Hand Knob Screws.
3. Attach the other (B2) U-Shaped Support Bar to the exposed screwholes on the vertical section of (A1) using 2 (D) Hand Knob Screws, with (B2) 's curves down toward the ground.



4. Attach optional (C) Magazine Basket to either side of the frame with armrest.



INSTALLATION INSTRUCTIONS

Once fully assembled, position the product up against the toilet so the center supports are wrapping around the base of the toilet. The armrests are located on either side for support when sitting and standing up.

SPECIFICATIONS

- Width: 26.5"
- Height: 25.5"
- Depth: 19"
- Width Between Armrests: 21"
- Product Weight: 9 LBS
- Max User Weight: 300 LBS

WARNING ⚠

- Check periodically to make sure all fixings are tight and secure before every use.

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
vhealth.link/24771 for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com